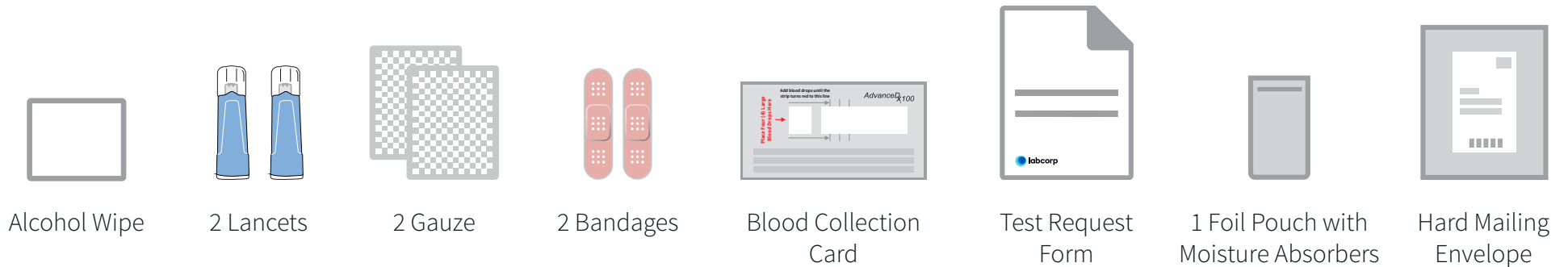
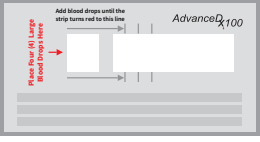
Blood Collection Instructions

Let's get started.

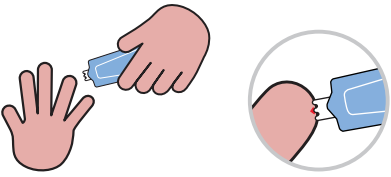
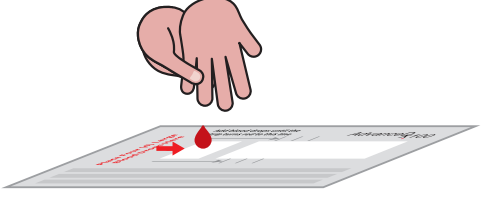
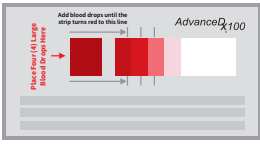
Please read instructions completely before beginning sample collection.

Kit includes:

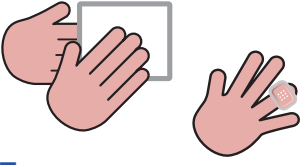
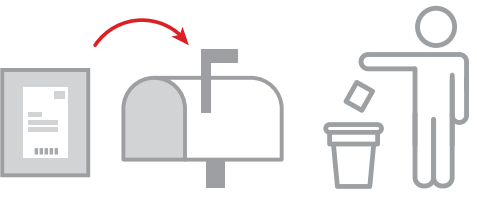
- 

Please print in ink

 - Place all kit contents on a clean, dry surface
 - Complete all fields as indicated on the Test Request Form. Please print in ink.
 - Where indicated, print your full name on the Blood Collection Card as it appears on the Test Request Form.
- 
 - Wash your hands with soap and warm water and dry thoroughly.
 - Rub your hands together for 10 to 15 seconds to help increase blood flow.
 - Remove the sterile gauze pad from the package and set it aside.
 - Lay the Blood Collection Card on the solid surface with the signed side of the card facing up.
 - Remove one of the blue-and-white lancets from the package.
- 
 - Remove alcohol wipe from the package and clean the tip of either your middle or ring finger with the swab. Allow your finger to air-dry for 10 to 15 seconds.
 - Twist and pull the white cap off of the lancet.
- 
 - Rest your forearm on a solid surface such as a countertop or table. With your palm facing up, place the tip of the lancet slightly off the center of the fingertip you cleaned.
 - Hold the lancet between your thumb and forefinger. To activate, press the lancet firmly against the fingertip. Do not remove the lancet until you hear a “click.” Then release the lancet.
- 
 - Hold the finger you pricked with the lancet about an inch above the Blood Collection Card. Gently massage your finger to bring a blood drop to the fingertip.
 - Without touching your finger to the Blood Collection Card, allow 4 large blood drops to fall on the left side of the Blood Collection Card as indicated. For some patients, this may take more than 4 drops.
 - Note:** Each lancet can only be used once.
- 

Dry for 1 hour

 - Keep adding blood drops in the first box only, until the red color spreads to the first line as indicated. (This may require more than 4 drops.)
 - Important:** Do not touch your finger to the Blood Collection Card.
 - If more blood is needed, repeat steps 3-6 with another finger.
- 
 - Wipe your finger with the alcohol wipe or another clean wipe, such as a tissue or cotton ball. Firmly hold the sterile gauze pad on your finger until the bleeding has stopped.
 - Place the provided bandage on your finger.
 - Allow your blood samples to air-dry on the Blood Collection Card for 1 hour. Do not place the card in direct sunlight while drying.**
- 
 - After your blood sample has dried for 1 hour, place the Blood Collection Card in the Foil Pouch with Moisture Absorbers and seal. Do not remove the moisture absorbers from the foil pouch.
 - Place the Test Request Form and the sealed pouch with your Blood Collection Card into the postage-paid, pre-addressed envelope and seal.
 - Discard all used and/or remaining materials. **Mail the envelope within 24 hours.**

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